


# Anti-Coagulative Effect of F202-Coated Polyurethane Catheters

**Control  
(uncoated)**

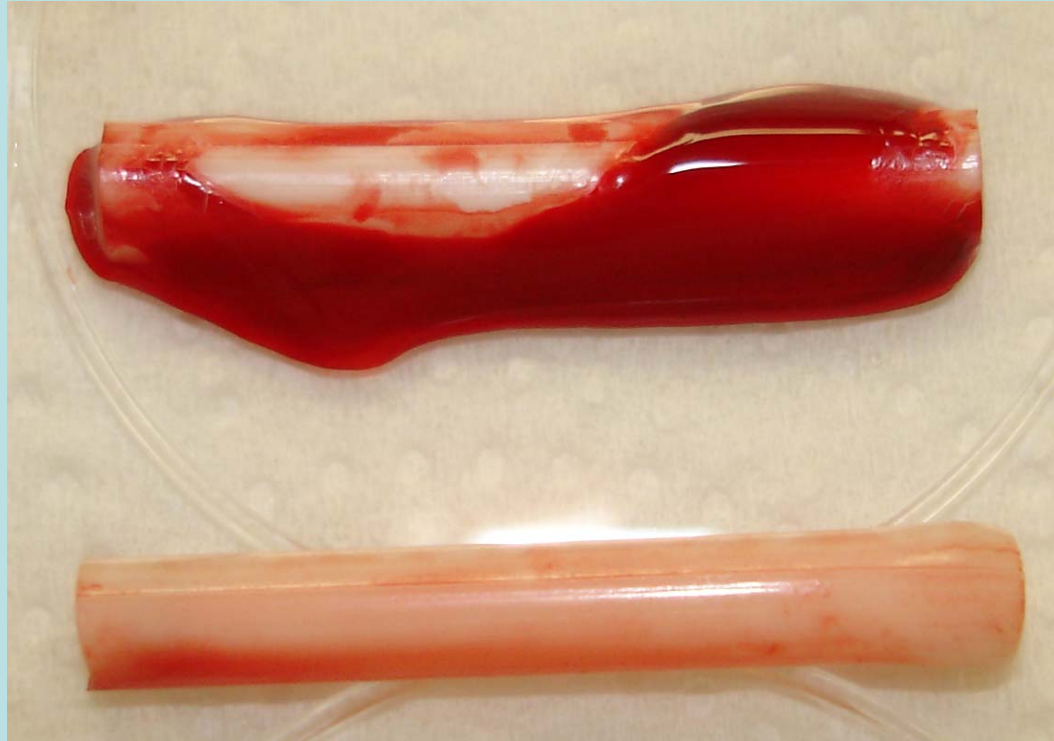

**Clot Time**

**< 30 minutes**

**F202-Coated**

**> 120 minutes**

A one inch section of uncoated and coated (outside only) catheter was placed in wells of a 6-well tissue culture plate. 1.5 ml of citrated human whole blood was recalcified (0.01M  $\text{Ca}^{2+}$  final) and incubated for 15 minutes @ RT on a lab rocker. The rocker was stopped and the incubation continued until clot formation was observed.